## DEUTSCH

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**3. Low/High Frequencies:** duplex®, duplex® baby, duplex® neonatal, tristar®, ri-rap® By turning the chest-piece, the membrane (high frequencies) or the bell side (low frequencies) can be set. When performing auscultation, it is essential to ensure that the surface adaptation is good. When measuring blood pressure, it is possible to push the chest-piece partly under the cuff with the membrane side showing downwards. It is possible to recognise immediately on the closed/opened bell outlet which side is active. After putting on of the binaural and slightly stroking across the membrane, it is possible to tell acoustically whether the membrane is active. With **ri-rap®** model it is also possible to change the small membrane side alternatively to an open bell. The different bell sizes are to be found in the box with the accompanying replacement parts.

cardiophon 2.0, duplex® 2.0, duplex® 2.0 baby, duplex® 2.0 neonatal: Our new models do have a special marking on the topside of the connection showing the user which membrane is currently open.

**4. Adjustment/Putting on of Binaurals** duplex®, duplex® baby, duplex® neonatal, tristar®, aneostophon®, ri-rap® Please ensure that the binaural are adjusted to the anatomically correct angle, i.e. the binaural are to be worn such that they slightly sit in front (see Fig. A).

The angle can be adjusted by turning the binaural individually. The ear-tips sit correctly if they rest completely at the same angle as the external auditory canal and disturbing external noises are optimally blocked out. The soft **Riester** ear-tips are particularly well-suited for this reason. In case the contact pressure of the binaural is too strong or too weak, proceed as follows: In case contact pressure is too weak, take both binaural and press them together, as shown in Fig. B. In case contact pressure is too strong, take both binaural and expand them, as shown in Fig. C.

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**ACHTUNG!** Die Ohrhüllen dürfen nicht zu tief in das Ohrkanal eindringen.  
**5. Exchanging the membranes:** duplex®, duplex® baby, duplex® neonatal, tristar®, aneostophon®, ri-rap® To change the membrane, take the non-chill ring between thumb and index finger and pull it off the chest-piece. Place the non-chill ring with the flat side facing downwards on an even surface and place the membrane into the non-chill ring with the printed side facing upwards. Take the non-chill ring with the membrane and push it onto the chest-piece. For the chest-piece of **ri-rap®**, please unscrew the ring nut, put the membrane in and fasten the ring nut again. If membranes with a bulge are used, the bulge needs to point away from **ri-rap®** chest-piece.

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#### WARRANTY

This product has been manufactured under the strictest quality standards and has undergone a thorough final quality check before leaving our factory.

We are therefore pleased to be able to provide a warranty of **2 years from the date of purchase** on all defects, which can verifiably be shown to be due to material or manufacturing faults. A warranty claim does not apply in the case of improper handling.

All defective parts of the product will be replaced or repaired free of charge within the warranty period. This does not apply to wearing parts.

For **r1 shock-proof** we grant an additional warranty of 5 years for the calibration, which is required by CE-certification. A warranty claim can only be granted if this Warranty Card has been completed and stamped by the dealer and is enclosed with the product.

Please remember that all warranty claims have to be made during the warranty period.

We will, of course, be pleased to carry out checks or repairs after expiry of the warranty period at a charge. You are also welcome to request a provisional cost estimate from us free of charge.

In case of a warranty claim or repair, please return the **Riester** product with the completed Warranty Card to the following address.

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**Rudolf Riester GmbH**