




Keeping Health Simple™

Order here!

PRAXISDIENST  
Medical Supplies since 1953

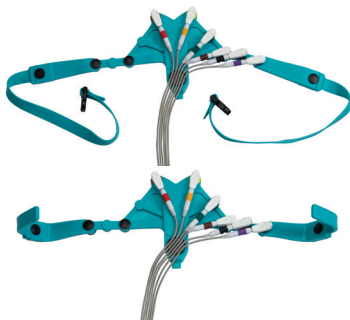



LevMed **AllBrand™** ECG Electrode Belt  
Ceinture d'électrodes ECG **AllBrand™** LevMed  
Cinturón de Electrodo ECG LevMed **AllBrand™**  
Cintura con elettrodi ECG **AllBrand™** di LevMed  
EKG-Elektrodengürtel LevMed **AllBrand™**

LevMed **AllBrand™** ECG-elektroderiem  
LevMed **AllBrand™** ECG Elektróda Öv  
LevMed **AllBrand™** EKG Elektrot Kemerü  
Ζώνη Ηλεκτροδίων ΗΚΓ LevMed **AllBrand™**

LevMed **AllBrand™** حزام إلكترود تخطيط القلب

O Produto: Cinta Elétrodo ECG, da Empresa, LevMed **AllBrand™**

LevMed **AllBrand™** EKG elektrodový pás

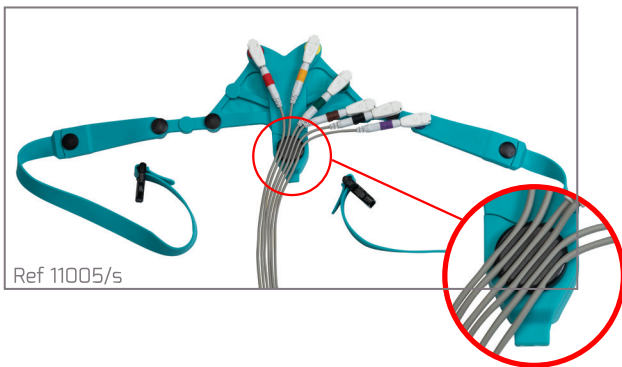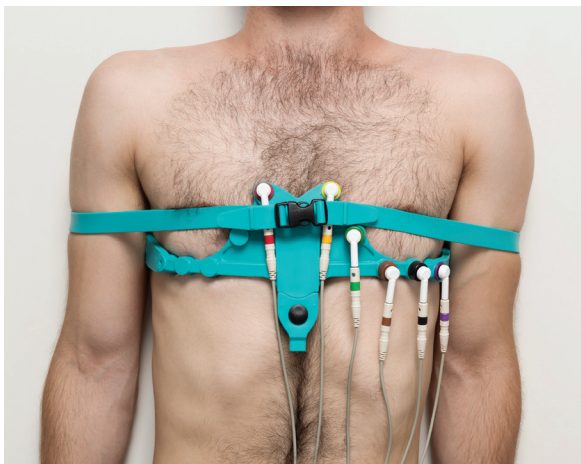



## INSTRUCTIONS FOR USE

### LevMed AllBrand™ ECG Electrode Belt



with Handles (Cat# 11005/h)



with Straps (Cat# 11005/s)



**Rx Only**

**Caution:** Read the complete list of instructions before using the product

**Caution:** Federal Law (USA) restricts this device to sale by or on the order of a physician

**INTENDED USE:** LevMed Allbrand™ ECG Electrode Belt is a reusable accessory intended to be used with ECG devices for short-term 12-lead rest ECG recordings for adults and pediatrics.

#### WARNINGS AND CAUTIONS

- Do not use this device for any purpose other than the stated intended use.
- Remove device before performing defibrillation.
- LevMed ECG Belt is intended to be used for rest ECG recordings, make sure the patient is relaxed, lying still, on a flat surface.
- Stay away from devices that create strong electromagnetic fields.

#### CONTRAINDICATIONS

- Do not use LevMed ECG belt on irritated or damaged skin.

#### SYMBOL LEGENDS



Caution, refer to instructions for use



Date of manufacture

**SN**

Serial number



Non Sterile

**REF**

Catalogue number



WEEE- Waste of electrical and electronic equipment



Manufacturer





## LevMed AllBrand™ ECG Electrode Belt

### INSTRUCTIONS FOR USE

#### 1. DEVICE DESCRIPTION

LevMed AllBrand™ ECG Belt is a system of ECG electrodes cast in a hyper-flexible silicone that is similar to ECG V1-V6 pre-cordial electrodes. The Belt is flexible and can be stretched to fit all body shapes and sizes, men and women alike; when fitted to the chest, it stretches to the required proportions, and the distance between the electrodes is automatically set. The Belt can be connected to any standard ECG patient cable.

#### 2. APPLICATION INSTRUCTIONS

Scan the applicable QR code for the Application process of the AllBrand™ ECG Belt:



For AllBrand with Handles (Cat# 11005/h)    For AllBrand with Straps (Cat# 11005/s)

#### 3. CLEANING & DISINFECTION

Wipe the LevMed ECG Belt with disinfectant wipe after each patient. Use only disinfectants that are dedicated for use on medical devices.

**Caution:** Do not use disinfectants containing petrol.

---

### WARRANTY

LevMed Ltd. warranty applies to the silicone components of the ECG Belt for a period of 12 months from the date of purchase or 15 months from the date of production, whichever is shorter. Cables, internal wires, electrodes, plastic parts, and other parts integrated within the ECG Belt are covered by warranty for a period of 3 months from the date of purchase or 6 months from the date of production, whichever is shorter. Warranty does not apply to any accessories supplied.

This warranty does not include or cover malfunctions caused by unreasonable use, noncompliance with user and maintenance instructions, or damage caused by unauthorized or unqualified repairs.