


Thermoal® rapid/kids/kids flex

Compartimento de la pila
Compartimento da pilha
Θήκη μπαταρίας
Battery compartment
حجيرة البطارية

Display extragrande
Mostrador digital grande
Πολύ μεγάλη ψηφιακή οθόνη
Extra large digital display
وحدة عرض كبيرة بصورة إضافية

Sensor de temperatura
Sensor de medição
Αισθητήρας μέτρησης
Thermometric sensor
مستشعر القياس

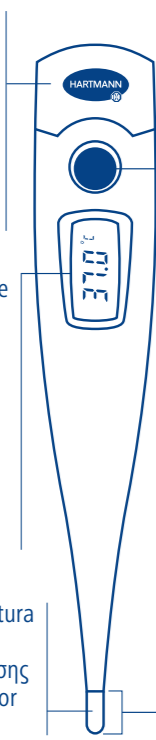

Botón de encendido/apagado
Tecla ON/OFF
Κουμπί ON/OFF
ON/OFF button
زر التشغيل/إيقاف التشغيل

Punta del termómetro
Ponta do termómetro
Μύτη θερμομέτρου
Thermometer tip
طرف الترمومتر


Thermoal kids flex:
con punta flexible
com ponta flexível
με εύκαμπτη μύτη μέτρησης
with flexible tip
بطرف قياس مرن

Normal temperature

36.6 – 38.0 °C

35.5 – 37.5 °C

34.7 – 37.3 °C

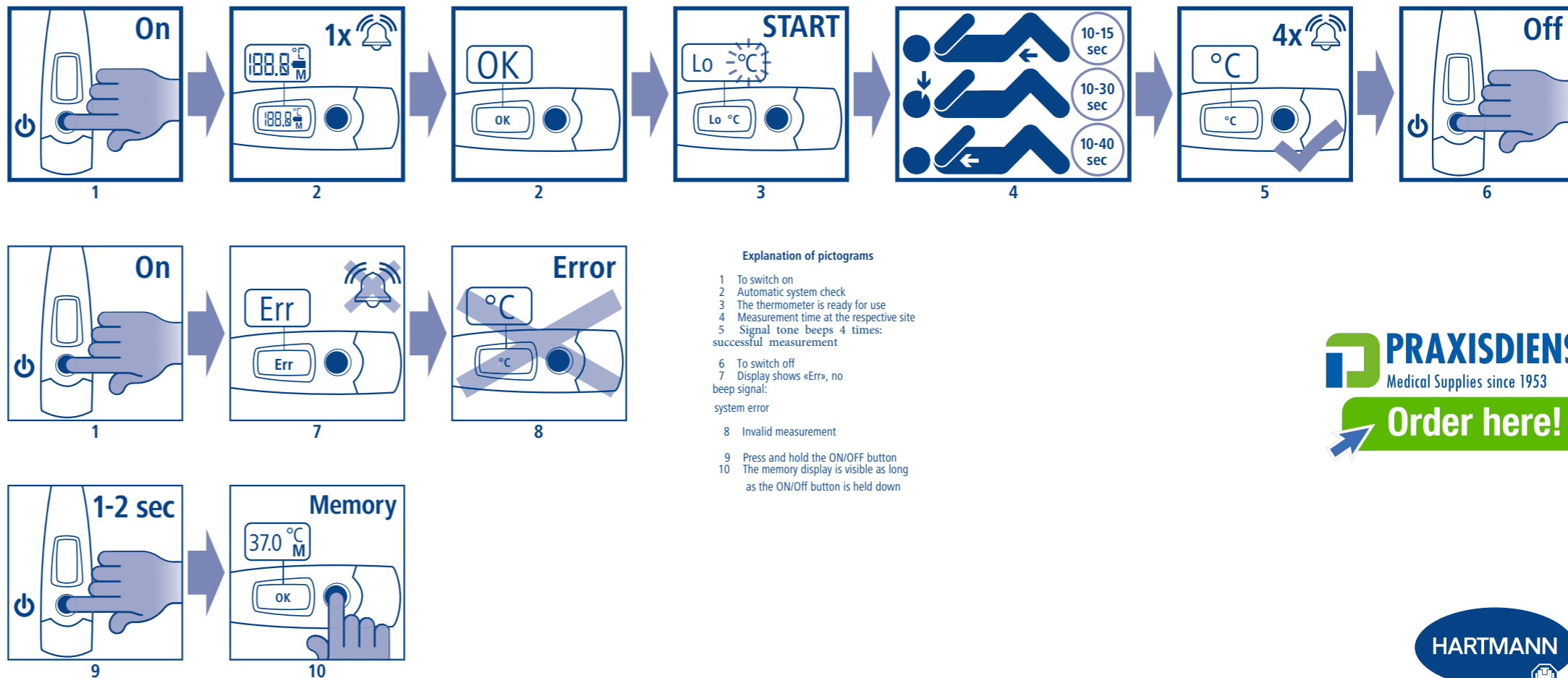

- Explanation of pictograms**
- To switch on
 - Automatic system check
 - The thermometer is ready for use
 - Measurement time at the respective site
 - Signal tone beeps 4 times: successful measurement
 - To switch off
 - Display shows «Err», no beep signal: system error
 - Invalid measurement
 - Press and hold the ON/OFF button
 - The memory display is visible as long as the ON/OFF button is held down

PRAXISDIENST
Medical Supplies since 1953
Order here!



