








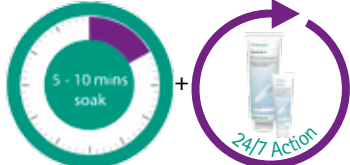

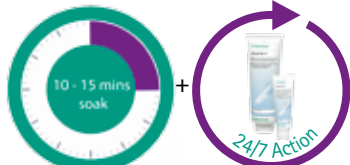
# Prontosan<sup>®</sup>

## BREAKS THE BIOFILM CYCLE

A proactive approach using a combination strategy of Prontosan Solution and Prontosan Gel X as part of wound bed preparation may prove helpful and aims to:

- Reduce the biofilm burden (**Prontosan Solution**)
- Prevent reconstitution of the biofilm (**Prontosan Gel X**)

### Appropriate Time for Appropriate Wound

DESCRIPTION OF WOUND	OBJECTIVE	HOW TO USE
<p><b>ACUTE WOUND - SURGICAL PRIMARY &amp; SECONDARY INTENTION HEALING</b></p>  <ul style="list-style-type: none"> <li>■ High risk patient*</li> <li>■ No slough</li> <li>■ Minimal exudate</li> </ul>	<ul style="list-style-type: none"> <li>■ Cleans</li> <li>■ Prevents biofilm / complications</li> </ul>	<p>Rinse with Solution</p> 
<p><b>ACUTE WOUND e.g. trauma</b></p>  <ul style="list-style-type: none"> <li>■ Debris</li> <li>■ Haematoma</li> </ul>	<ul style="list-style-type: none"> <li>■ Cleans</li> <li>■ Prevents biofilm / complications</li> </ul>	<p>Soak with Solution</p> 
<p><b>CHRONIC WOUND - GRANULATING</b></p>  <ul style="list-style-type: none"> <li>■ High risk patient*</li> <li>■ Low exudate</li> </ul>	<ul style="list-style-type: none"> <li>■ Cleans</li> <li>■ Prevents biofilm / complications</li> </ul>	<p>Soak with Solution Consider Gel X</p> 
<p><b>CHRONIC WOUND</b></p>  <ul style="list-style-type: none"> <li>■ Light slough</li> <li>■ Low exudate</li> </ul>	<ul style="list-style-type: none"> <li>■ Cleans</li> <li>■ Prevents biofilm / complications</li> </ul>	<p>Soak with Solution Apply Gel X</p> 
<p><b>CHRONIC WOUND - CRITICALLY COLONISED/INFECTED</b></p>  <ul style="list-style-type: none"> <li>■ Medium / high exudate</li> <li>■ Static wound</li> <li>■ Slough</li> </ul>	<ul style="list-style-type: none"> <li>■ Cleans</li> <li>■ Prevents biofilm / complications</li> </ul>	<p>Soak with Solution Apply Gel X</p> 

\*High risk patient: Co-morbidities such as Diabetes, immuno-compromised, steroidal use, patients with previous wound infection and or biofilm and slough.

Compatible with other dressings | Once opened, Prontosan Solution and Gel X can be used for up to 8 weeks (single patient use)