TRUglue®

Single Dose

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Product description:
TRUGLUE® is a one-component adhesive which is only intended for the closure of the upper skin or mucosa of humans or animals.

Areas of application:
Tissue adhesive for external use and for the application in the oral cavity and at the external genital area. For the application to fresh lacerations, abrasion and surgical incisions and also for protection/closure after removal of skin lesions (nevi, warts, moles, senile freckles and many other more). For the closure of wounds after careful suturing of the wound edges, as a suture substitute, for additional suture support or as an anti-infective protection for sutured wounds.

Contraindications:
Do not use on infected or inflamed or extensive (abraded) wounds or on wounds from bites. Do not use to close wounds of inner organs, the brain surface or the central or peripheral nervous system for TRUGLUE® is not absorbable. An application to these areas might damage the tissue and the subsequent scarring may cause complications. The application to the intima and media of blood vessels should be avoided, because vascular thrombosis and damage of the vascular wall may occur. The tissue adhesive may come off if applied to extremely sweating patients. Do not use for patients with a known hypersensitivity to cyanoacrylate or formaldehyde (also see: Side effects).

Precaution for the application and warnings:
Adheres skin and eyelids within seconds. Avoid any contact with eyes and eyelids! TRUGLUE® must not come in contact with the eyeball or conjunctival sac. Apply to external wounds, the oral mucosa and external genital area only. Ensure the exact adhesion of wound edges prior to application. Surfaces to be adhered should be dry and free of grease as possible. Liquid or pasty agents must not be applied before adhering. Do not drip adhesive into the depth of the wound, but apply in thin layers to skin surface. Apply to fresh wounds only (not older than 6 hours). Do not use on contaminated or inflamed wounds. Do not inhale fumes! Do not apply TRUGLUE® to the breasts during lactation period.

Hint:
The vapours which arise during the polymerization of the adhesive will irritate the conjunctiva of the eye and the mucous membrane of the nose. Therefore, we recommend protecting the patient's eyes and nose with humid cotton wool when wounds in the face are closed.

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Side effects:
If a "throbbbing" feeling appears a few hours after the wound has been adhered, it is advised to peel off the adhesive and to treat the wound otherwise. It might be assumed that at the time of adhesion the wound was already inflamed or foreign bodies were included.

If it should be noticed during a check-up that not enough TRUGLUE® was applied and, therefore, the material peeled off the surrounding healthy skin, another layer can be applied without any problem, because TRUGLUE® provides no culture medium for germs, and even has an inhibitory effect on existing germs. The TRUGLUE® solution is sterile (Medical Device Services May 2008 / MedCert October 2008).

After 5 to 10 days when the healing process is advanced, TRUGLUE® detaches itself from the skin like scab. A further treatment of the wound is mostly not necessary.

Storage instructions and shelf life of TRUGLUE®:
Follow the storage instructions on the package and the single bags. The best way to store TRUGLUE® is in the deep freezer or at least in the refrigerator. TRUGLUE® can be used immediately after removal from the deep freezer, but it does not freeze into a solid block. The expiry dates are mentioned on the aluminum pouches as well as on the outer package. Do not use TRUGLUE® after these dates. Discontinue use if the solution does not drip in a clear watery consistency any longer. The material has then started to polymerize and an exact application is no longer possible. The adhesive must not be used any longer.

Important: Apply the adhesive sparingly and spread it out thinly. It is not advisable to apply one thick layer, as this does not improve the adhesive strength, but causes the adhesive to peel off the healthy skin more easily. Avoid contact of the adhesive with gloves, instruments, gauze, etc. to prevent adhesion.

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TRUGLUE® provides no culture medium for germs and even has an inhibitive effect on existing germs.

Important notes for the use of Ethyl-2-Cyanoacrylate Wound Adhesive TRUGLUE® in the Dental Practice:
TRUGLUE® es un adhesivo de etil-2-cianoacrilato para un solo uso suministrado en aplicadores monodosis con 0,3 ml de contenido.

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
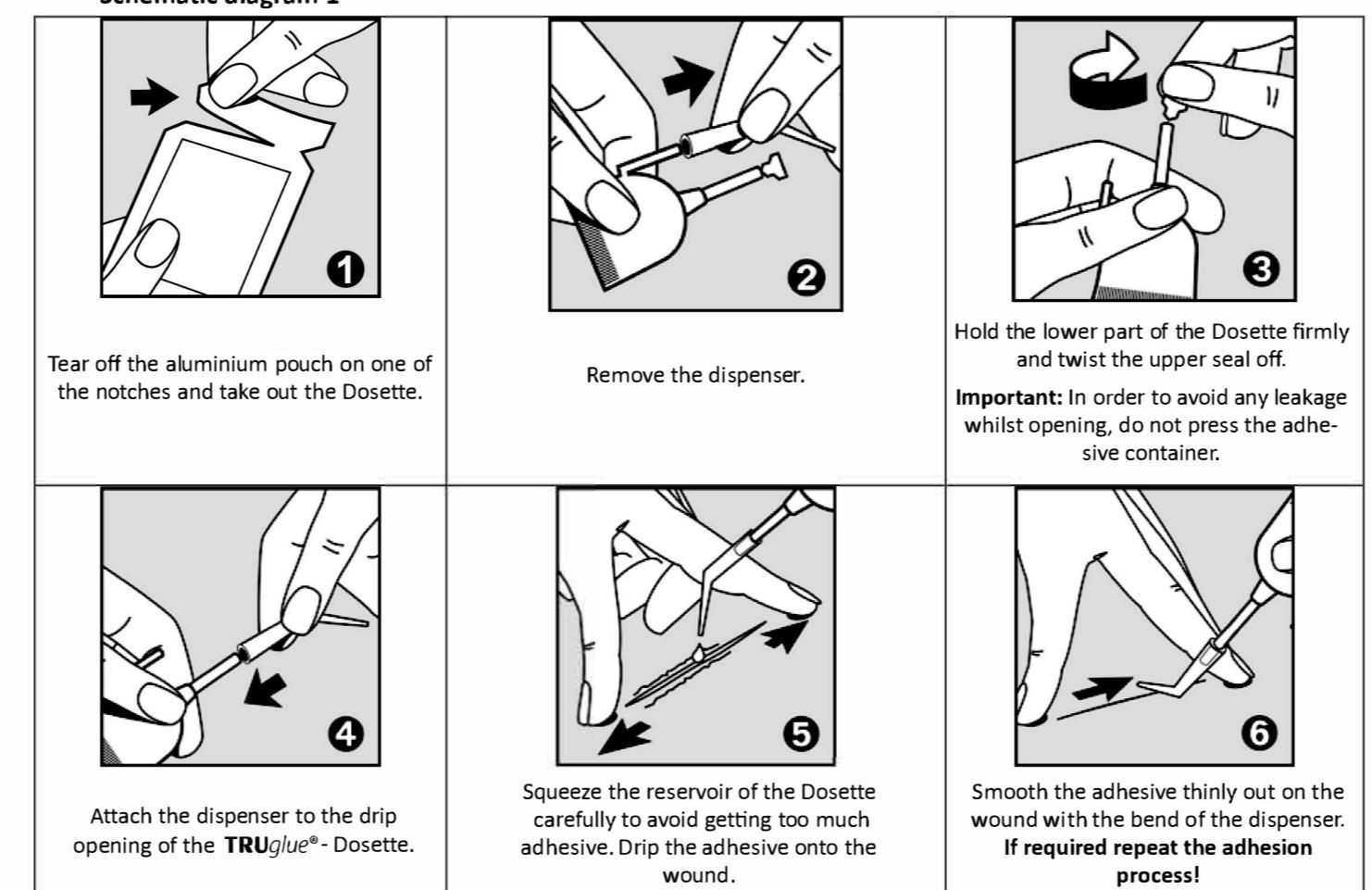
To Re-Order

TRUglue®-Single Dose
Package with 5 Dosettes each containing 0.3 ml
Package with 10 Dosettes each containing 0.3 ml
Package with 25 Dosettes each containing 0.3 ml

Schematic diagram 1

Schematic diagram 2

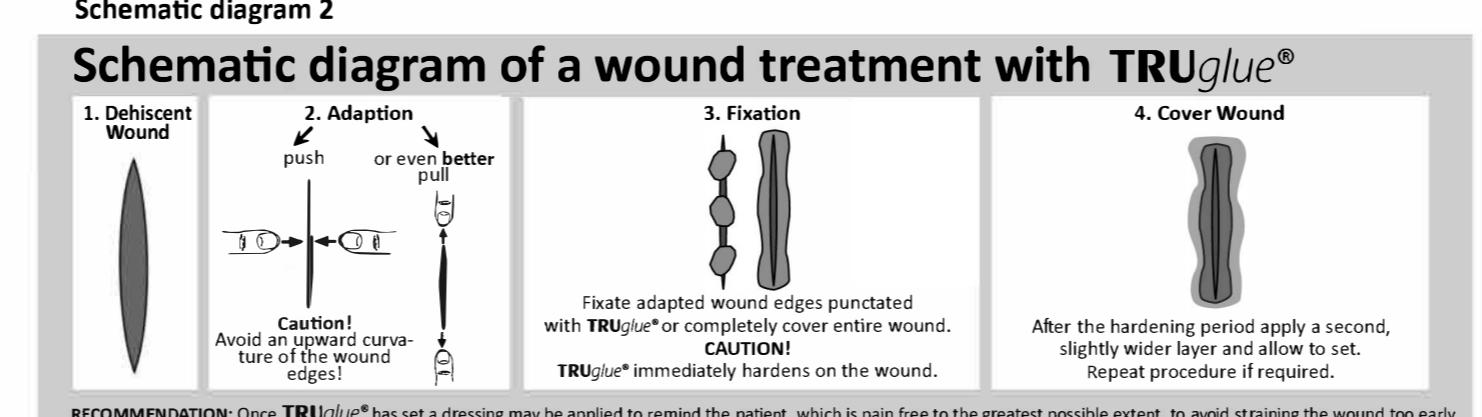Now the wound is closed secure of infection. The patient is immediately able to resume his personal daily hygiene like washing and showering (not too hot). Please advise the patient not to rub the place treated with TRUGLUE® while foaming and wiping off but only dabbing it carefully. It is recommended not to take a full bath or a sauna.

Notes for health professionals:

Hazard statements:
Causes skin irritation.
Causes serious eye irritation.
May cause respiratory irritation.

Precautionary statements:
Avoid breathing in vapours.
Wear protective gloves / protective clothing / eye protection / face protection.

Storage:
Store in a well-ventilated place. Keep container tightly closed.

Disposal of contents / container:
Wash thoroughly after handling.
Use only outdoors or in a well-ventilated area.

Distribution:
Distributor

Manufacturing and control:
Manufacturer

CE 0482 CE-mark and identification number of the Notified Body

Limit of temperature

Code of lot

Precaution

Information for use

Warning

Do not re-use!

Temperature limit

Lot / Batch

Storage:

Keep TRUGLUE® locked up.

Disposal of contents / container in accordance with local / regional / national / international regulations.

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