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7083753 Harvard TraySive, 10 ml Flasche

**Harvard TraySive**  
**Tray adhesive**

Harvard TraySive is a universal tray adhesive for silicones based impression materials.

Harvard TraySive ensures a reliable adhesion of Harvard PremiumSil impression material to the impression tray and avoids uncontrollable deformation during impression taking.

**Indications**

Harvard TraySive secures **safe adhesion** between Harvard PremiumSil impression materials and metal- or methacrylate trays.

**Note:**

Harvard TraySive does not always secure sufficient adhesion to trays made of other plastic materials or individual trays made of base plates. A polyether adhesive must be used for polyether impression materials.

**Working Instructions**

Apply a thin layer of Harvard TraySive to the tray using the applicator brush and allow to dry at least **3 minutes**. Then load the tray and take the impression.

**Cleaning from the tray**

The impression material can be removed with a blunt instrument.

Soaking in a universal commercial solvent or in petroleum benzine will dissolve Harvard TraySive. After that the tray can be cleaned and disinfected as usual.

**Warning: Use solvents only in well ventilated areas.**

**Warning**

Harvard TraySive should only be used in well ventilated rooms. In case of contact with skin, wash off with water.

**Storage information**

Do not store above 25 °C (78 °F). Do not use after expiry date.

**Warranty**

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental Internationals sole obligation shall be repair or replacement of the Harvard Dental International product.

**Limitation of Liability**

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children!**  
**For dental use only!**

**Caution:**

**Federal law restricts the sale of this device to or by the order of a dentist.**

The instructions for use has to be kept for the duration of the application.

**Order-No. Article**

7083753 Harvard TraySive, 10 ml bottle

Symbols used in international labeling

Consult the instructions for use

Lower and upper temperature limits

Expiry Date (JJJJ-MM)

Lot number

Manufacturer

For single use only

Protect from humidity

Protect from sunlight

Order-No.

Restricted to sale on the order of a dentist

 <p>Manufacturer</p>	<p><b>Harvard Dental International GmbH</b>  <b>Margaretenstr. 2 – 4</b>  <b>15366 Hoppegarten, Germany</b></p>	 <p>Made in Germany</p>	<p>Tel: +49 (0) 30/99 28 978-0          Fax: +49 (0) 30/99 28 978-19          info@harvard-dental-international.de          www.harvard-dental.de</p>	<p>V01.1 10/2016          10960</p>
---	---	--	---	---