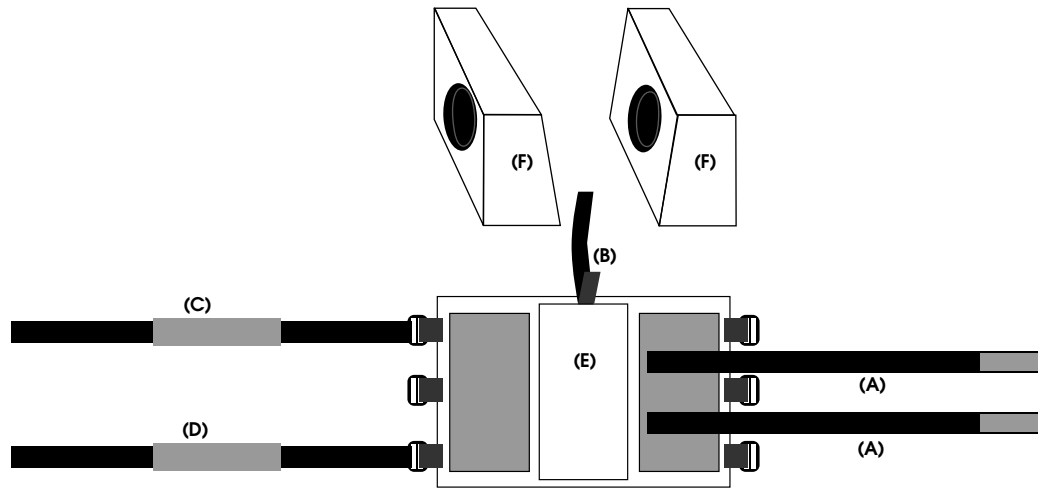
ultraMEDIC GmbH & Co. KG
Im Bruch 11-15
D-56567 Neuwied, Germany
Phone:
+49 (0) 263-196-9830
Fax:
+49 (0) 263-196-98333
www.ultramedic.de

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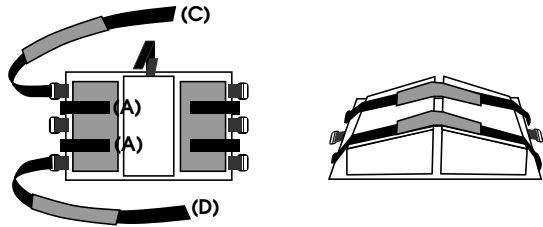
Components

- (A) Fastening straps with velcro
- (B) Upper retaining strap with velcro
- (C) Forehead strap with padding and velcro
- (D) Chin strap with padding and velcro
- (E) Base plate with padding and large velcro areas
- (F) Support cushion with large velcro area;
1 inclined side and 1 straight side

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Preserving/Storage

ultraHEAD-LOCK can be folded up into a compact package and stowed in a space-saving manner. Thread the retaining straps (A) below the base plate and fasten them with the velcro straps. Place the support cushion on the base plate and fasten it with the headband (C) and chin strap (D) through the D-rings. This method of storage also facilitates fast fixing of the ultraHEAD-LOCK.

Inspection and care

Regular inspection of the ultraHEAD-LOCK is extremely important. When doing so, look for anything that might affect the problem-free working of the head immobilisation fitment, like loose velcro straps, torn or frayed fabric. These inspections must be carried out at least once a month; in case of continuous use, correspondingly more frequently. Any head immobilisation set with signs of intense wear must be replaced immediately.

ultraHEAD-LOCK should be cleaned regularly with hot water and/or mild cleaning agents that do not attack the material.

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OPERATING MANUAL

ultraHEAD-LOCK is a universal head immobilisation fitment for scoop stretchers and spine boards. ultraHEAD-LOCK consists of soft foam with velcro straps for fixing. The material is PVC-free, easy-care and completely washable. The ears remain uncovered, X-rayable.

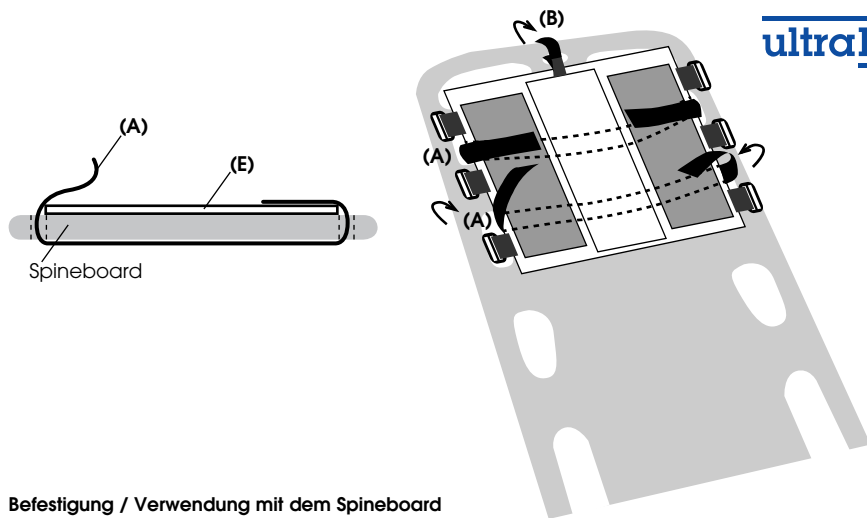
Head immobilisation

ultraHEAD-LOCK head immobilisation fitment serves for the additional immobilisation of the head when there is a suspicion of injuries to the cervical spine.

The side openings in the middle of the supporting cushion simplify communication between paramedic and patient. It is also possible to check whether there is any fluid discharge from the ears.

NOTE

Be extremely careful when applying the ultraHEAD-LOCK. When taking care of a patient with a suspicion of cervical spine injuries, dispensing with additional immobilisation aids can result in further injuries.

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
Fastening / Using with the spine board

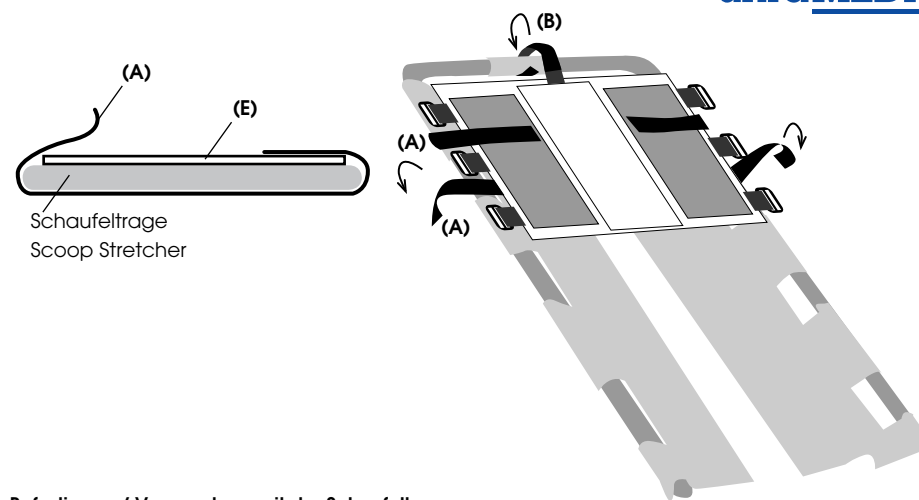Dismantle the head immobilising fitment into its individual components. Place the base plate (E) at the head end of the spine board. For fastening the base plate, the retaining straps (A) are threaded through and pulled out on both sides of the head part and fastened on the velcro strap of the base plate. The upper retaining strap (B) is threaded from the top through the hole at the end of the head piece and secured (see the diagram below).
 In case of a head injury and/or a cervical spine injury, a cervical spine support collar is generally wrapped before the patient is moved. Moving is done by the usual methods employed by the emergency services, and will not be dealt with here. Then, the supporting cushions are fastened, with the straight side next to the head of the patient on the large velcro plate.
 Now put the headband (C) over the forehead of the patient (velcro side upwards). In doing so, the padded middle part should lie uniformly on the forehead. For fastening, the two ends of the head band are threaded through the D-rings located on the base plate, carefully pulled tight and fastened. The strap length should be corrected if required.
 The position of the headband depends on the type of injury. Decide, depending on the conditions, whether applying the headband is necessary. Also with regard to the chin strap (D), check whether it is necessary to place it over the chin and the cervical spine supporting collar.

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Fastening / Using with scoop stretchers

In case of a suspicion of a cervical spine injury, generally, a cervical spine supporting collar is inserted, and then the scoop stretcher continues to be used.
 Once the patient is lying on the scoop stretcher, a helper supports the head and the neck of the patient. The second helper then carefully pushes the base plate at the head of the stretcher below the head of the patient. The base plate is fastened with the two fastening straps (A) to the scoop stretcher. The upper retaining strap (B) is placed from the outside to the inside around the tubular frame of the scoop stretcher in the area of the lock and secured (see the diagram below).
 Then, the supporting cushions are fastened with the straight side next to the head of the patient on the large velcro plate.
 Now place the headband (C) over the forehead of the patient (velcro side upwards). The padded middle portion should then be uniformly on the forehead. For fastening, the two ends of the head band are led through the D-rings located on the left and right of the base plate, pulled tight carefully and fastened. The strap length should be corrected if required.
 Removing the strapping is generally done under the instructions of a doctor after the X-ray or the diagnosis.